


Follow these instructions to properly position your D100 headgear and mask. The device must be turned on in order for the mask to seal to the face properly. There should be minimal or no leaks. **NOTE:** *Refer to instruction manual for more detailed instructions.*

STEP 1




Place headgear with Velcro side down on a flat surface. Detach two Velcro tapes on A1 & A2.

STEP 2

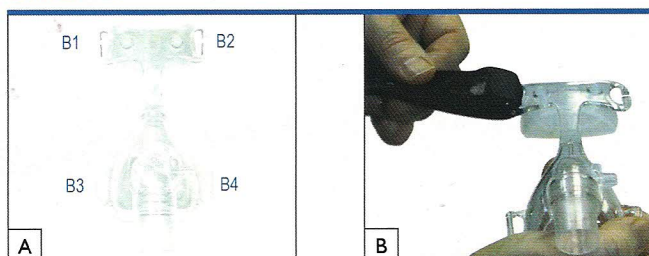


Identify headgear fastening positions on the mask. Pull the upper headgear strap A1 through the slot B1 on the mask, making sure the Velcro faces outward. Fasten the Velcro tape.

STEP 3

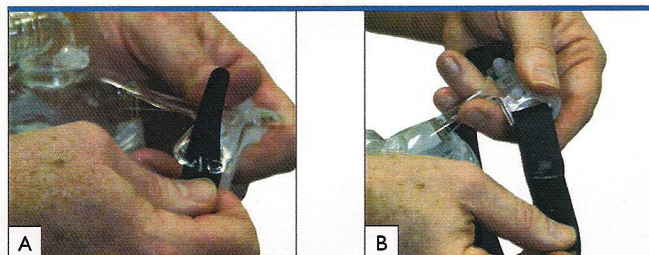


Pull the upper headgear strap A2 through the slot B2 on the mask, making sure the Velcro faces outward. Fasten the Velcro tape.

STEP 4

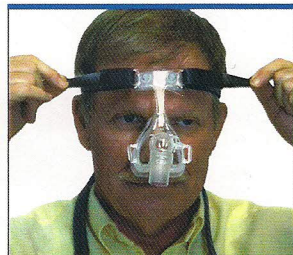


Place the headgear over your head and adjust both upper headgear straps to a comfortable tension.

STEP 5

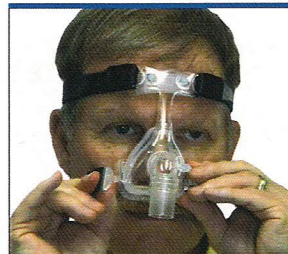


Hook the plastic buckle A3 onto B3 and A4 onto B4 on the mask frame.

STEP 6

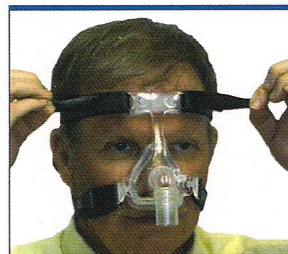

Adjust again both upper headgear straps A1 & A2 to a comfortable tension.

STEP 7

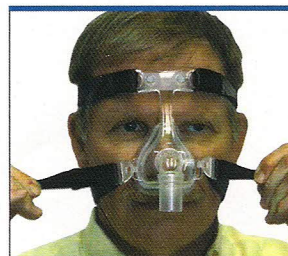

Again, adjust both lower headgear straps A3 & A4 to a comfortable tension.

STEP 8



Ensure the overhead strap is in proper position to avoid slipping down.