


3.30"

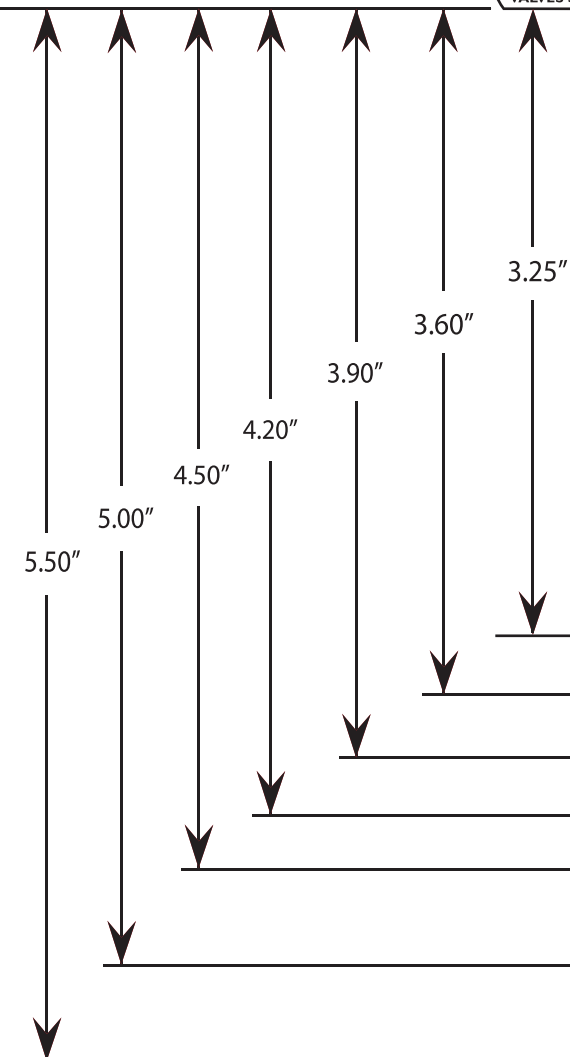

MAKERS OF RESPIRATORY VALVES SINCE 1938

HRI

MASK SIZING GAUGE

For Hans Rudolph, inc.
Oro-Nasal Vmask™ and V2mask™
Series Face Masks

Directions: Length measurements are used to determine the size of the Hans Rudolph Oro-Nasal mask that will best fit the patient. For measurement: the patient's facial muscles should be relaxed and jaw closed. Place the Gauge Tip to the deepest point of the nasal root: depression (sellion) and note the size indication at the bottom tip of the chin. The color band widths indicate the size ranges. (This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.)

3.25"

3.60"

3.90"

4.20"

4.50"

5.00"

5.50"

7.25"

If between sizes, we recommend you use the smaller size.



PETITE

EXTRA-SMALL

SMALL

MEDIUM

LARGE

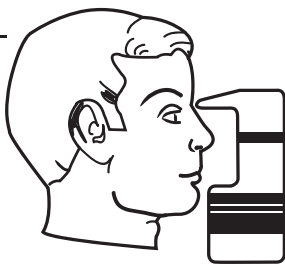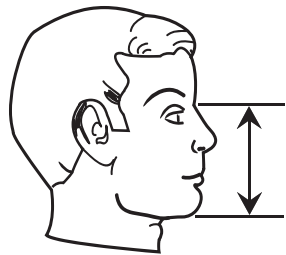

HANS RUDOLPH, inc.
MAKERS OF RESPIRATORY VALVES SINCE 1938

ISO 9001 ISO 13485

Tel: (913) 422-7788 USA & CANADA (800) 456-6695

Fax: (913) 422-3337 E-mail: hri@rudolphkc.com

www.rudolphkc.com

8325 Cole Pkwy, Shawnee, Kansas 66227 USA



Printed in USA
691143 5/06 Rev B

2.75"