




# PAD A CHEEK<sup>®</sup>

## Forehead or Buckle Pad Style B Fits MOST MASKS

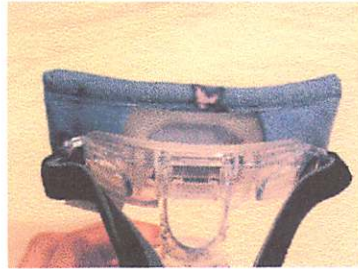

This package contains 2 Forehead Pads and one Free additional padding piece.

### FOREHEAD PAD

To install a **PAD A CHEEK- Forehead** on your mask:  
Slide the pocket on one end of the **PAD A CHEEK** over the end of the forehead support on the mask. Repeat the same procedure for the other side of the forehead piece.

NOTE: If the straps on your mask attach to the forehead pad, you can put the straps through the holes cut in the end of the **PAD A CHEEK** pad. Smooth and straighten the **PAD A CHEEK** until it lies flat and will be comfortable against your forehead.



### BUCKLE



To install **PAD A CHEEK-Buckle** :

Remove the strap from one side of the buckle.  
Insert the strap thru the hole in the edge of the pad. (Material will stretch but you may want to enlarge the hole slightly.)  
Re-attach the strap to the buckle.  
Repeat these steps for the other buckle strap.  
Smooth the **PAD A CHEEK-Buckle** so it lies flat.  
You can leave the strap ends out so you can adjust them if necessary without removing the pad.

This **PAD A CHEEK-** is made from 100% polyester micro fleece, and can be machine or hand washed with warm water. Do not use bleach. **PAD A CHEEK-Buckle** is machine or air dryable (low heat).

**PAD A CHEEK** 434-985-4003  
157 Sunset Drive, Stanardsville, VA 22973

email [admin@padacheek.com](mailto:admin@padacheek.com)

web site [www.padacheek.com](http://www.padacheek.com)

Designed and sewn in the USA