

GET YOUR PATIENTS STARTED ON PROVENT THERAPY

Introducing a new starter kit to get your patients successfully acclimated on Provent Sleep Apnea Therapy. This comprehensive system supports your patients during the acclimation phase, by gradually increasing resistance in the first 4 nights.

Clinical study data has demonstrated that patients may benefit from a longer and more gradual adjustment period to Provent Therapy.

We are proud to introduce a new system to maximize success with Provent Therapy.



Provent Therapy Starter Kit

Provent Therapy uses the patient's own breathing to create EPAP (expiratory positive airway pressure) to treat OSA (obstructive sleep apnea). Patients adapt to breathing with Provent at their own pace. The Starter Kit's convenient 3-phase approach is specifically designed to help the patient adjust. Clinical research supports the use of a gradual increase in resistance to enhance acclimation. Patients reported having an easier time falling asleep and sleeping better while using the Starter Kit.

The Starter Kit includes a 4 night adjustment period.

- **Phase 1** (Nights 1 & 2) and **Phase 2** (Nights 3 & 4). Lower, non-therapeutic resistances.
- **Phase 3** (nights 5-30) Full therapeutic resistance.



Get Started Prescribing

The simplicity of Provent Therapy makes it easy to implement in your practice. To accurately assess the effectiveness of Provent Therapy for your patients, we suggest a few simple steps:

- Review Provent Instructions for Use with the patient
- Try sample device on patient in the office
- Send prescription to your local supplier
- Schedule a follow-up appointment with your patient in 2-4 weeks

Prescribing Provent Sleep Apnea Therapy

1. Complete prescription form
2. Fax Prescription form to Provent Therapy supplier of your choice
 - For a list of Provent Therapy suppliers, go to www.proventtherapy.com/supplier_locator

PROVENT SLEEP APNEA THERAPY	
Rx	PRESCRIPTION
<input checked="" type="checkbox"/>	Provent Therapy Starter Kit (one month) Qty: No Refill
<input checked="" type="checkbox"/>	Provent Therapy 30-night refill
Rx Duration	
<input type="checkbox"/>	3 months
<input checked="" type="checkbox"/>	12 months



EFFICACY CONFIRMATION

PROVENT THERAPY WITH THE NASAL CANNULA KIT

Ongoing clinical assessment of the use and effectiveness of nasal EPAP therapy should be conducted by the healthcare professional. This may include:

- Subjective evaluation to assess compliance and symptom response (eg, Epworth sleepiness scale or other questionnaire).
- Objective evaluation of efficacy ideally using in lab PSG or portable monitoring.

The Provent Nasal Cannula Kit

- A diagnostic tool to be used with the Provent Sleep Apnea Therapy device during polysomnograms (PSGs) or home sleep testing to confirm product efficacy.
- Allows transmission of pressure signals from the nose to a pressure transducer of a physiologic recorder.
- Single use device only



Ordering Information

The Provent Nasal Cannula Kit is available through MVAP Medical Supplier

Toll free: **1-877-735-6827** (MVAP)

Toll free fax: **1-877-735-7213**

To order online: **www.mvapmed.com**

Search keyword: "Provent"