EXPION360

THE LITHIUM PROS

EXP Pro Series Hand-held Power Banks EXP48Pro & EXP96Pro

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3.0 MAIN ROUND POWER PORT(S)

EXP48PRO(1)/EXP96PRO(2)

12V @ 10A Continuous / 15A Peak for 5 minutes (5 min).

4.0 MAIN POWER PORT(S) OPERATION

- A. Plug your device into the main round power port ensuring a secure fit, using a 12V cigarette style plug.
- B. The port(s) will automatically turn on and provide power to the connected device.
- C. To turn off the device, simply unplug it from the power port.

5.0 CHARGE STEPS

- A. Select the method of charging. AC, DC (Included) or Solar (Sold Separately).
- B. Plug the charging cable into the charge input port and the other end into the power source.
- C. Charging will begin automatically.

6.0 CHARGE TIME

EXP 48PRO: 4-6 hours ± @ 2A.

EXP 96PRO: 10-12 hours ± @ 2A.

7.0 OTHER IMPORTANT CHARGING INFORMATION

- A. Only use the supplied AC and DC charge cord to charge the battery bank. Use of a different charge cord other than the two supplied may damage the battery bank.
- B. Never connect the input charge port and the round main output power port with the provided DC charge cord on the same battery bank. Doing so may damage the battery.
- C. Never leave the battery bank unattended for long periods of time while charging. When charging is complete disconnect the battery bank charge cord from the charge source.
- D. Do not use the AC or DC charge cords (included) to power a PAP machine or other appliance. This may damage the PAP machine, appliance or the battery bank.
- E. Do not charge below 32°F. Optimum ambient charge temperature is 40°F - 110°F.
- F. Do not charge the battery bank if the charge cord or battery bank has obvious damage or if the battery bank appears to be bulging. A bulging battery bank is a potential sign of damaged lithium cells.
- G. Do not charge the battery bank if it feels excessively hot. The battery bank should not reach more than 120°F while charging or discharging and should not feel hotter than a warm cup of coffee to the touch.

8.0 BATTERY MANAGEMENT SYSTEM(BMS)

The battery bank has a built-in BMS that controls balanced charging and discharging of the individual Lithium cells contained within.

It also provides protection from:

- Over-Charging.
- Over-Discharging.
- Over-Current.
- Over-Temperature
- Short Circuit

If the BMS has been tripped to protect the lithium cells and has shut down the battery bank simply plug in the AC or DC charge cord as if charging and the BMS will reset itself and resume normal operation.

9.0 BATTERY BANK FEATURES AND SPECIFICATIONS

EXP 48PRO / EXP 96PRO

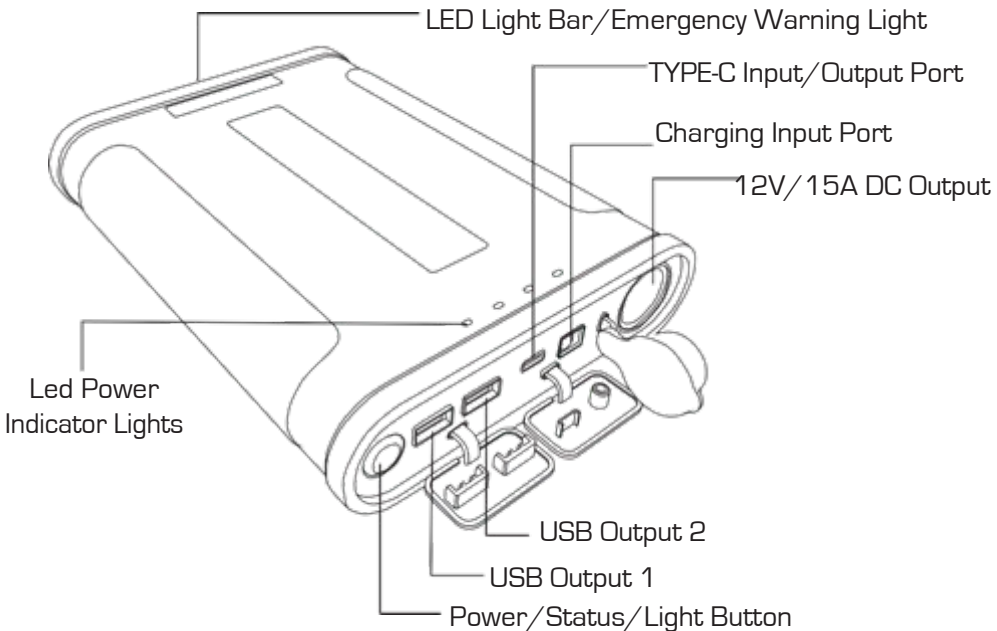
Main DC Round Output Power Port: 12V@10A Continuous/15A Peak (5Min)

Type C Input DC: 5V@3A / 9V@3A / 15V@3A / 20V@2.25A.

Type C Output DC: 5V@3A/9V@3A/12V@3A/15V@3A /20.3V@3A

USB 1 DC Output: 5V@2.4A.

USB 2 DC Output: 3.6V-6.5V@3A / 6.5V-9V@2A / 9-12V@1.5A

10.0 PAP MACHINE PORTABLE POWER FOR SLEEP APNEA

There are two methods of powering a PAP machine using the EXP Series Battery Bank.

1. Using A DC Cord. Using the DC power cord specific to the individual PAP machine being powered (sold separately), plug one end of the DC power cord into the PAP machine and the other end into the main round power port marked 12V/15A. Expion360 PAP machine DC Power Cords are available for the most popular PAP machines and are listed below. Other makes and models are available through the PAP machine manufacturer and supplier.
2. Using A Inverter. Plug the AC power cord included with the PAP machine into the AC outlet on the inverter and plug in the cigarette style plug into the main power port marked 12V/15A on the battery bank. The inverter will convert DC power from the battery to AC power required for the factory AC power cord.

11.0 PAP MACHINE DC POWER CORDS

ResMed Air 10 Series: Part No. RM-10

ResMed S9 Series: Part No. RM-9

Respironics DreamStation Series: Part No. DS-12V Respironics
System One REMstar 60 Series: Part No. S1-60

Please visit our website for a list of sub-models our 12V DC power cords will work with.

www.expion360.com

12.0 PAP MACHINE RUN TIMES

To determine the hours of run time you will get using your PAP machine connected to the EXP Series Battery Bank you will need to refer to your PAP machine manufacturer's specifications to determine the amount of amperage your machine uses at different pressure settings. To get the most run time from the battery bank we recommend turning off all auxiliary features such as humidity and heated hose if possible. These features use a large amount of amperage and will draw most of the available power from the battery bank. PAP machine nightly run times listed below for the EXP 48PRO and EXP 96PRO are figured using the average night sleep in the US of 7 hours and with auxiliary features turned off. Power requirements will vary greatly from one PAP machine to the next and with different prescribed pressure settings. We recommend you test your PAP machine with the battery bank prior to depending on it for emergency or off-grid power. It is important that your mask fits properly. Air leakage will result in higher power consumption and reduced run times. Unplug your PAP machine from the battery bank when not in use, to conserve power.

EXP 48Pro 2-4 Nights

EXP 96Pro 4-6 Nights

13.0 PARALLEL CONNECTING THE EXP SERIES BATTERY BANKS

You can parallel connect no more than two battery banks by plugging in the provided DC charge cord to the charge port on one battery bank and the main power port marked 12V/15A on the other battery bank in order to double the available capacity.

14.0 AIRLINE TRAVEL WITH THE EXP SERIES BATTERY BANKS

Check current FAA regulations and the individual airline rules prior to airline travel. Do not put the EXP Series battery bank or any other lithium batteries or devices containing lithium batteries in checked baggage of any type. If allowed by FAA regulations and individual airline rules only take in carry-on baggage. Current FAA regulations can be found on their website at www.faa.gov search "Pack Safe Lithium Batteries". The major airlines list rules for lithium batteries on their websites.

15.0 SAFE USE OF LITHIUM BATTERIES

- Never expose the battery banks to liquids, especially saltwater. This can cause internal corrosion and damage to the lithium cells and or the BMS, causing them to malfunction.
- Never drop the battery banks. This can damage the lithium cells, or more likely the BMS, causing them to malfunction.
- Never operate the battery banks in direct sunlight. This is hard on the BMS, electrical components and the lithium cells. Prolonged use in direct sunlight can damage the battery bank.

16.0 WARRANTY

We provide a full replacement of the battery bank and the AC or DC charge cords, if defective, within a 2 year period after date of purchase. Please retain your original sales receipt for proof of purchase and return to your original retailer.

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S9 ESCAPE CPAP WITH ERP,
S9 ELITE CPAP WITH ERP,
S9 AUTOSET CPAP, S9 VPAP S BILEVEL

DS-12V

PHILIPS RESPIRONICS DREAMSTATION DC POWER CORD

CABLE DE ALIMENTACIÓN DE CC PHILIPS PARA RESPIRONICS DREAMSTATION
CORDON D ALIMENTATION CC DU MOD LE P ILIPS RESPIRONICS DREAMSTATION

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DREAMSTATION CPAP, DREAMSTATION PRO,
DREAMSTATION AUTO, DREAMSTATION BIPAP AUTO

S1-60

PHILIPS REMSTAR60 SERIES DC POWER CORD

CABLE DE ALIMENTACIÓN DE CC PHILIPS SERIE REMSTAR 60 SERIE
PHILIPS REMSTAR 60 CORDON D ALIMENTATION CC

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The Expion360 30W Folding Solar Panel is the perfect accessory to take your hand-held EXP 48Pro on the go. This panel is specifically regulated to charge the EXP 48Pro quickly while you are still on the move. Contact your local Expion360 Dealer for more details.

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