OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

1. Connect the large end of the Connector (T) to the Swivel (J) on the mask and the small end of the Connector to the main breathing tube (R).
2. Open the Port Cap(s) (P) and place the pressure and/or oxygen tube (S) firmly over the Port(s) (Q).

(The two ports are identical; either port may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

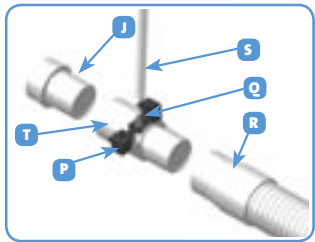
- **Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

- **Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

- **Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- **Warning:** Do not use Polyvinyl Chloride (PVC) pressure and/or oxygen tubing. Use silicone tubing instead.

Explanation of Warning: Contact with PVC can cause premature cracking of ports.

TECHNICAL SPECIFICATIONS

EXHAUST FLOW

Nasal masks for CPAP and Bi-Level have exhaust flow holes to expel the air that you exhale from the mask. The FlexiFit™ 405 Nasal Mask has a unique Diffuser system that is designed to diffuse the air that is expelled from the mask. This reduces the draft.

The Diffuser does not significantly change the amount of air flowing from the mask, therefore the mask can be used with or without the Diffuser.

PRESSURE (cmH ₂ O)	3	5	7	9	11	13	15	17	19	21	23	25
FLOW (L/MIN)	18	24	29	33	37	41	44	47	50	53	55	58

RESISTANCE TO FLOW

Pressure drop through mask @ 50 l/min 0.1 cm H₂O

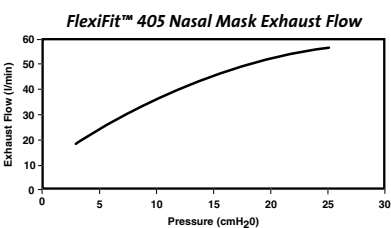
Pressure drop through mask @ 100 l/min 0.5 cm H₂O

MASK DEAD SPACE

88cc

OPERATING CONDITIONS

- Operating pressure is 3-25 cm H₂O
- Operating temperature range is 5-50°C (40-120°F)

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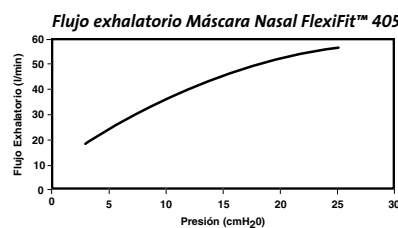
MULTI-PATIENT CLEANING INSTRUCTIONS

Your FlexiFit™ 405 Nasal Mask is provided patient-ready. To reuse between patients please follow the guidelines below.

CLEANING / DESOILING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THE DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH-LEVEL DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

MASK BASE 	AUTOCLAVING Autoclave 15 minutes' exposure at 121°C (250°F). OR THERMAL DISINFECTION Soak parts in hot-water bath for 30 minutes at 70°C (158°F). OR STERAD® 100S Follow manufacturer's instructions for Sterrad® 100S sterilization system. OR CIDEX® OPA Follow manufacturer's instructions for high-level disinfection. Immerse for 12 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR SPOROX® II Follow manufacturer's instructions for high-level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
SILICONE SEAL 	
ELBOW 	
SWIVEL 	

HEADGEAR 	Hand-wash the Headgear in pure soap dissolved in lukewarm water. Dry out of direct sunlight.
---------------------	--

FOAM CUSHION 	Replace with new part.
-------------------------	------------------------

DIFFUSER 	
DIFFUSER COVER 	If continuing use with the diffuser, replace with a new part.

THE FLEXIFIT™ 405 NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

Disclaimer: It is the users responsibility to qualify any deviations from the recommended method of reprocessing.

Contact your local Homecare Provider for additional information.

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SWEDEN
Tel: +46 8 564 76 680
Fax: +46 8 36 63 10

SWITZERLAND
Tel: 0800 83 47 63
Fax: 0800 83 47 54

UK
(EU Authorised Representative)
Fisher & Paykel Healthcare Ltd
Unit 16, Cordwallis Park, Clivemont Road, Maidenhead,
Berkshire SL6 7BU, United Kingdom
Tel: +44 1628 626 136
Fax: +44 1628 626 146

Northern Ireland
Tel: +0800 132 189

USA
Tel: +1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

CE 0123

© Fisher & Paykel Healthcare Limited

Fisher & Paykel
HEALTHCARE

FLEXIFIT™ 405 NASAL MASK PATIENT INSTRUCTIONS FOR USE

THE FLEXIFIT™ 405 NASAL MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT.

FITTING YOUR MASK

1. Remove plastic cover from Silicone Seal.
 2. Holding your mask in one hand, spread the Headgear (K) with your free hand, just enough to fit comfortably over your head.
 3. Lightly hold the mask against your face, covering the nose, then slowly pull the Headgear over your head.
 4. Attach the Headgear to the Glider™ Strap (E).
 5. Pull the lower straps as low as possible down the back of your neck. Gently tighten straps to remove slack. Connect to CPAP/Bi-Level machine at prescribed pressure.
- If necessary, tighten the lower straps to eliminate any leaks (avoid over-tightening the straps).
 - Use the Silicone Seal that provides the best fit for you. Small and Large are both provided with your mask.
 - When you need to remove your mask, unhook the Headgear from the Glider™ Strap.

FITTING TIPS

- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
- If there are any leaks after gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. This will allow the Silicone Seal to inflate with air. Put the mask back onto the face.

CLEANING YOUR MASK

To ensure the continuing safety and comfort of your FlexFit™ 405 Nasal Mask:

- **BEFORE FIRST USE:**
 1. Hand wash the Headgear (K) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes then rinse in fresh water and allow to air dry out of direct sunlight.
- **BEFORE EACH USE:**
 1. Inspect the mask for deterioration. Do not use if damaged.
- **AFTER EACH USE:**
 1. Disassemble the Silicone Seal (C), Foam Cushion (D), Elbow (B) and the Headgear (K) from the Mask Base (A).
 2. Wash the Silicone Seal and Mask Base in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water.
 3. Wipe the Foam Cushion with a damp cloth. (Do not immerse the Foam Cushion in water).
 4. Leave all parts to dry out of direct sun light before reassembling. The Diffuser (F) should not be washed or rinsed. Simply replace it with a new Diffuser if it becomes noisy or deteriorates. It should last two to three months. Small dots may appear on the Diffuser - these do not affect its performance. The FlexiFit™ 405 Nasal Mask may be used with, or without the Diffuser and Diffuser Cap (G).
- **EVERY 7 DAYS:**
 1. Hand wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to air dry out of direct sunlight.

CAUTIONS

- Do not soak the mask for longer than 10 minutes.
- Do not wash the mask in the dishwasher.
- Do not clean the mask with products containing alcohol, antibacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ 405 Nasal Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your FlexiFit™ 405 Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for reassembly.

REFER TO THE MASK ASSEMBLY DIAGRAM.

1. Place the Foam Cushion (D) into the Mask Base (A).
2. Starting at the apex of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom of the mask.
3. Push the Elbow (B) into the Mask Base.
4. Lay the Headgear (K) flat, with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (I) and the Glider™ Strap (E). This can be done without undoing the Velcro™ tabs, by sliding the Headgear into the slots.

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly.
- **Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks which have exhaust flow holes to allow continuous flow of air out of the mask. When the CPAP/Bi-Level machine is turned on and functioning properly, new air from the CPAP/Bi-Level machine flushes the exhaled air out through the mask's exhaust flow holes. However, when the CPAP/Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocations.
- At low CPAP/Bi-Level pressures the flow through the exhaust flow holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Do not block the exhaust flow holes or use alternative Diffusers, other than those manufactured by Fisher & Paykel Healthcare. Before use, always ensure that gas is flowing out the exhaust flow holes or through the Diffuser (if it is being used).
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If re-using the FlexiFit™ 405 Nasal Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the 405 Nasal Mask.

CAUTIONS

- Use the mask for its intended use only, as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of, a physician.

The FlexiFit™ 405 Nasal Mask and accessories are Latex free. Many people find CPAP/Bi-Level therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms.

US PATENT NUMBER 6 615 834
Other Patents Pending
FOR ADDITIONAL INFORMATION CONTACT YOUR LOCAL HOMECARE PROVIDER

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