



PAD A CHEEK®

Anti-Leak Strap

AirFit F20

and AirTouch F20



**Do not use on the Quattro Air, Air Fit F10, Mirage FX,
or any mask where it would cover the vents**

- Gently pull the hose off the hose coupling at the mask. Slide the hole in the **Anti-Leak Strap** completely over the elbow coupling and slide it down flush to the mask.
- **Do not cover any of the exhaust holes. Do not use on the AirFit F10**
- Wrap the strap around your head and attach the ends together.
- There is a long piece of Velcro in the back you can uncover if you wish to secure the strap to your headgear.

This **PAD A CHEEK® Anti-Leak Strap** is made from 100% polyester micro fiber. It can be machine or hand washed with warm soapy water and air or machine (low heat) dryable.

PAD A CHEEK® 434-985-4003

Email: admin@padacheek.com

157 Sunset Drive, Stanardsville, VA 22973

website: www.padacheek.com

