

## CUSTOMIZING THE COMFORTGEL CUSHION

**Important!** The headgear must be partially disconnected before you customize the gel cushion.


1. **Prepare the Mask and Water.** Disconnect the neckstraps by removing the swivel clips from their sockets. Fill a bowl with enough cold water to cover the entire mask cushion. SET ASIDE.

Fill a pan with enough water to cover the entire mask cushion. Place the pan over medium heat, and bring the water to a boil. REMOVE THE PAN FROM THE HEAT.


**NOTE: DO NOT allow headgear to get wet!**  
**CAUTION! DO NOT place Gel Cushion into Microwave!**

**CAUTION!** DO NOT Microwave the Gel Cushion.

**WARNING!** Water and contents may be hot.

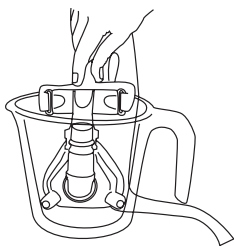

**Soak mask in hot water for 4 minutes.**

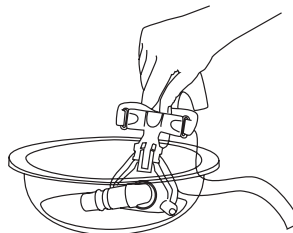

**Fill bowl with cold water and cover the entire mask cushion.**

2. **Warm the Cushion.** Hold the mask by the headgear and place the mask cushion into the hot water. Let the mask soak for 4 minutes. Do not allow the headgear to get wet!
3. **Cool the Cushion.** Remove the mask from the hot water, and place it in the cold water. Let the mask cool for 10 seconds. Remove the mask from the cold water and shake any excess water from the mask.

**IMPORTANT!** Touch the mask cushion to be sure that it has cooled enough to prevent irritation or burning.

4. **Re-form the Cushion.** Position the mask over your nose, and press the mask cushion gently against your face. Pull the headgear over your head, and connect the swivel clips.