

RESMED

AutoSet™ & Elite™

POSITIVE AIRWAY PRESSURE DEVICES



Welcome Guide

English

S9™ Series
More. Comfort.

S9[™] | Series
More. Comfort.

Welcome

Thank you for choosing the S9 AutoSet or S9 Elite.
Before operating these devices, please read the entire
Welcome and Information Guides.




S9 at a glance

The S9 system comprises the following elements:

- S9 device • SlimLine air tubing • 90W or 30W power supply unit • S9 travel bag • SD card
- S9 SD card protective folder.

Optional components include:

- H5i heated humidifier • H5i water tub • Standard air tubing • ClimateLine heated tubing
- 90W power supply unit • 30W power supply unit (does not support H5i).

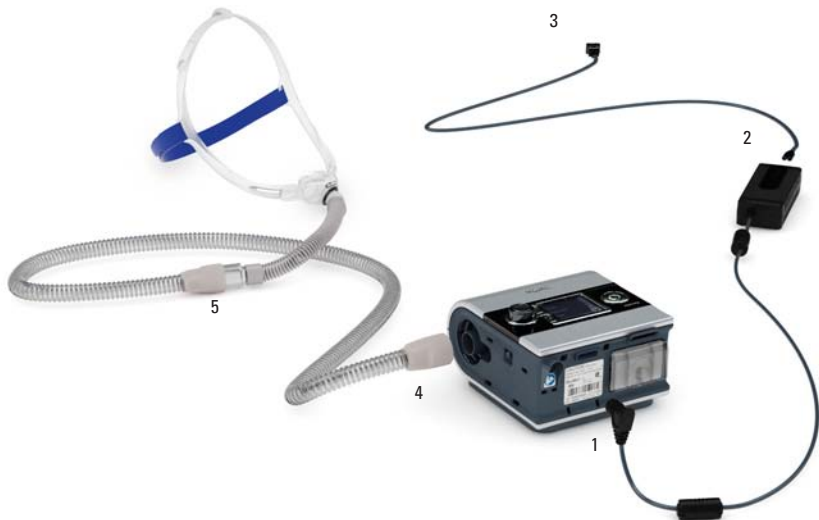

Setup

1. Connect the DC plug of the power supply unit to the rear of the S9.
2. Connect the power cord to the power supply unit.
3. Plug the other end of the power cord into the power outlet.
4. Connect one end of the air tubing firmly onto the air outlet.
5. Connect the assembled mask system to the free end of the air tubing.

Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on www.resmed.com on the Products page under Service & Support.

Start/Stop button
Starts or stops
treatment.

Home menu

LCD screen

Info menu
Allows you to
view your sleep
statistics.

Setup menu
Allows you to
make changes
to settings.


Push dial
Turning the dial allows
you to scroll through
the menu and change
settings. Pushing the
dial allows you to enter
into a menu and confirm
your choice.

S9 basics


The control panel of your S9 device includes an LCD screen as well as a Start/Stop button, Push dial and the Info and Setup menu keys.


Key:


Home 


Humidity level 


Ramp 


Heated tube 

Climate Control 

Start/Stop 

Push dial 


Setup menu 





Info menu 

RESMED

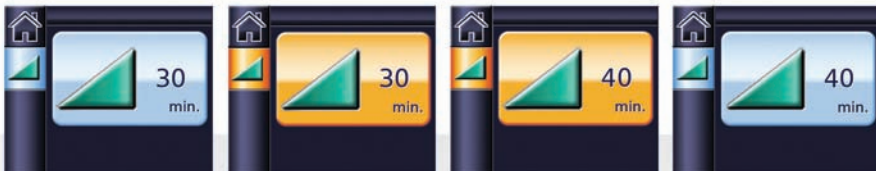Adjusting ramp time


Designed to make the beginning of treatment more comfortable, ramp time is the period during which the pressure increases from a low start pressure to the treatment pressure. To adjust your ramp time:




1. From your home screen, turn  until the RAMP screen is displayed in blue.
2. Press . The selection is highlighted in orange.
3. Turn  until the ramp time you require is displayed.
4. Press  to confirm your choice.

Using mask-fit

You can use mask-fit to help you fit your mask properly. This feature delivers treatment pressure for a three-minute period, prior to starting treatment, during which you can check and adjust your mask-fit to minimise leaks.

1. Fit your mask as described in the mask user guide.
2. From your home screen, push and hold  for three seconds. One of the following MASK FIT screens is displayed:



3. If necessary, adjust your mask, mask cushion and headgear until you have a Good mask-fit. After three minutes, the pressure reverts to the set pressure and treatment will begin. You can end mask-fit at any time by pressing .


Selecting the mask type

To select your type of mask:



1. Press . The SETTINGS screen is displayed.




2. Turn  until Mask is displayed in blue.




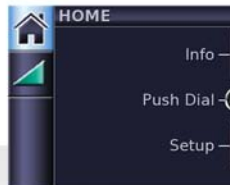
3. Press . The selection is highlighted in orange.




4. Turn  until the mask setting that you require is displayed.





5. Press  to confirm your choice.



6. Push  to return to the home screen.



Getting started

1. Make sure the power is connected.
2. Adjust the ramp time if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy, simply breathe into the mask and/or press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time, remove your mask and/or press .

***Note:** If your clinician has enabled SmartStart your device will start automatically when you breathe into the mask and stop automatically when you remove the mask.*

Cleaning and maintenance

Refer to your mask and humidifier user guides for detailed instructions regarding the care of these devices.

Daily

Disconnect the air tubing and hang it in a clean, dry place until next use.

Weekly

1. Remove the air tubing from the S9 device and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang, and allow to dry.
4. Before next use, reconnect the air tubing to the air outlet and mask.


Monthly

1. Wipe the exterior of the S9 with a damp cloth and mild detergent.
2. Check the air filter for holes and blockage by dirt or dust.

Replacing the air filter

Replace the air filter every six months (or more often if necessary).

1. Remove the air filter cover from the back of the S9 device.
2. Remove and discard the old air filter.
3. Insert a new ResMed air filter ensuring that it is sitting flat in the air filter cover.
4. Replace the air filter cover.

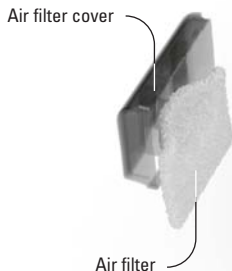


SD card

An SD card has been supplied to gather therapy data from your S9 device and provide settings updates from your clinician. When instructed to do so, remove your SD card, insert it in the protective folder and send it to your clinician.

Removing the card

1. Push in the SD card to release it.
2. Remove the card.
3. Insert the card into the protective folder.
4. Send the protective folder back to your clinician as instructed.

For more information on removing and inserting your card refer to the S9 SD Card Protective Folder provided with your device. Please retain the S9 SD Card Protective Folder for future use.

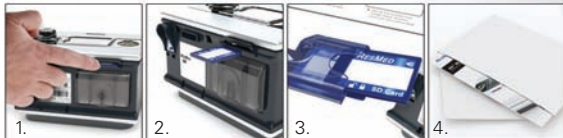

Notes:

- For Troubleshooting information refer to the Information Guide.
- For more product information see www.resmed.com.

S9[™] | Series
More. Comfort.

368125-Eng/2 09 10
S9 Series
AutoSet & Elite
WELCOME
EUR 1

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide. S9. Protected by patents: AU 691200, AU 697652, AU 702820, AU 709279, AU 724589, AU 730844, AU 736723, AU 750095, AU 750761, AU 764761, AU 779327, EP 0651971, EP 0661071, EP 0920845, EP 0927538, EP 0934723, EP 1026769, EP 1126893, EP 1144036, EP 1502618, JP 3580776, JP 3778797, NZ 504595, US 5199424, US 5245995, US 5522382, US 5704345, US 6029665, US 6138675, US 6240921, US 6363933, US 6367474, US 6425395, US 6502572, US 6591834, US 6675797, US 6745768, US 6817361, US 6988498, US 7040317, US 7100608, US 7320320, US 7537010. Other patents pending. Design registrations pending. S9 is a trademark of ResMed Ltd. H5i: Protected by AU 2002233025, CN ZL028049365, HK 1065483, NZ 527088, US 6935337. Other patents pending. Designs pending. © 2009 ResMed Ltd. 368125-Eng/2 09 10

